

Dental Dam

Instruction for Use

SANCTUARY HEALTH SDN. BHD.

Intended Use:

Dental dams are thin sheets of latex or non-latex used to isolate specific teeth during dental procedures.

The domain of usage of dental dam is to ensure the best use of dental treatment, their benefits includes:

Infection control barrier

Isolate a tooth to obtain a dry working field

Improve the operation way/ field

Patient management

Increase operational efficiency

Storage conditions:

Latex dental dam and non-latex dental dam is an easy to aging, and the condition of preservation of the product is very sensitive. Many variable factors (especially temperature) can affect this material. Therefore the storage temperature should be below 40°C (104°F).

Warranty Period:

In order to achieve the best usage effect, dental dam should be used before the expiry date as stated on colour box. Expiration of latex dental dams is 5 years of the manufacturing date while for non-latex dental dams is 3 years of the manufacturing date.

Duration of use:

It is intended for continuous use for less than 60 minutes during dental procedures.

Product Specification:**Dental Dams Product Specifications:**

Dental Dams	Dimension		Colour
Latex	127mm x 127mm (5" x 5") 152mm x 152mm (6" x 6")	Thin	Green/ Blue
		Medium	
		Heavy	
		Thin	Black
		Medium	

		Heavy	
		Thin	Silk Blue
		Medium	
Non-Latex	127mm x 127mm (5" x 5") 152mm x 152mm (6" x 6")	Medium	Purple

Basic principles which medical device works on and their features

- (i) Place the Dental Dam in a flat surface.
- (ii) Determine the center of dam and punch a hole approximately 2mm diameter. Choose the right size of puncher according to the contacted teeth.
- (iii) Use a forceps to enlarge the hole and place a metal clamp to hold the enlarged hole.
- (iv) Place a dam and the clamp into the affected tooth.
- (v) Place the dental dam frame outside the mouth to hold excess dam material.

Caution: This product contains natural rubber latex which may cause allergic reactions (It is only applicable for latex dental dam).

Manufacturer:

SANCTUARY HEALTH SDN. BHD.

Address: NO. 16, PERSIARAN PERINDUSTRIAN KANTHAN 1, KAWASAN PERINDUSTRIAN KANTHAN, 31200 CHEMOR.

E-mail: info@sanctuaryhealth.com.my

Website: <http://sanctuary-dental.com/>

Telephone: +60-5-201 2800

Fax: +60-5-201 7800