

Dental Dam Accessories

Instruction for Use

SANCTUARY HEALTH SDN. BHD.

Intended Use:

Dental Dam Clamp hold the dental dam in place to the cervical region for isolation from the rest of the exposed teeth.

Dental Dam Punch is used to obtain a perfect punch hole on dental dam using 7 different sized-holes. The tip of the punch creates well-formed hole in correspond with targeted tooth.

Dental Dam Forceps is used to position the dental dam clamp securely on and off the tooth.

Dental Dam Frame is used to hold the dental dam in a stretched state to provide increased visibility and access during operative procedure.

Storage conditions:

Store at room temperature.

Warranty Period:

NIL

Duration of use:

NIL

Product Specification:**Dental Dams Accessories Product Specifications:**

Product name	Product code	Product Description
Dental Dam Punch	RDPN1	Dental Dam Punch, 159mm (L)
Dental Dam Clamps	RDC210	Anterior for adult, 30.40mm (W) x 21.30mm (L)
	RDC211	Anterior for adult, 29.40mm (W) x 19.90mm (L)
	RDC207	Premolar for adult, 16.18mm (W) x 16.40mm (L)
	RDC2	Premolar for adult, 17.40mm (W) x 16.20mm (L)
	RDC2A	Premolar for adult, 17.38mm (W) x 18.30mm (L)
	RDC201	Molar for adult, 17.90mm (W) x 21.60mm (L)
	RDC202	Molar for adult, 18.00mm (W) x 24.10mm (L)
	RDC203	Posterior for children, 14.39mm (W) x 20.10mm (L)
	RDC204	Posterior for children, 14.39mm (W) x 20.10mm (L)
	RDSTD2	Dental Dam Clamps with Organizer, 3cm (H) x 10cm (W) x 15cm (L) Included Dental Dam Clamps: 1.) RDC210 2.) RDC211 3.) RDC207

		4.) RDC2 5.) RDC2A 6.) RDC201 7.) RDC202 8.) RDC203 9.) RDC204
Dental Dam Forceps	RDPL1	Dental Dam Forceps, 162mm x 48mm x 31mm (L)
Dental Dam Frame	RDFR1	Dental Dam Frame, 110.2mm (W) x 95.3mm (L)
	RDFR2	Dental Dam Frame, 135.7mm (W) x 102.5mm (L)

Basic principles which medical device works on and their features

Preparation for use:

1. Check the device to be in good condition.
2. This device is non-sterile.
 - a) Sterile the product accordingly.
 - b) During sterilization or drying procedure, secure enough space between instruments so that they do not come in contact with each other. A sterilized wrap must be used.

Method of use:

1. Punch a hole on the dental dam using a dental dam punch.
2. Consider the tooth size, location, gingival architecture and choose the right size of dental dam clamp.
3. Fit the dental dam onto the dental dam frame.
4. Fix the dental dam clamp firmly on the tooth with dental dam forceps.

Caution

1. Staining may be caused during sterilization. Follow the instructions provided. Do not open the case until the steam has completely evaporated.
2. Do not use the product other than its intended use.
3. Store the device at room temperature.
4. Sharp object may cause tear on the dental dam.

Manufactured for:

SANCTUARY HEALTH SDN. BHD.

Address: NO. 16, PERSIARAN PERINDUSTRIAN KANTHAN 1, KAWASAN PERINDUSTRIAN
KANTHAN, 31200 CHEMOR, MALAYSIA.

E-mail: info@sanctuaryhealth.com.my

Website: <http://sanctuary-dental.com/>

Telephone: +60-5-201 2800

Fax: +60-5-201 7800